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Instructions for use and care of AIREX® Balance-pad / Balance- pad Elite / Balance-beam

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Dear Customer

Congratulations on your purchase of a high-quality AIREX® brand product! To enjoy the benefits of our CE-conform AIREX® speciality foam products for as long as possible, please note these care instructions.

Use

AIREX® Balance products are ideal for co-ordination or balancing exercises. Prior to use the floor should be tested for **glide resistance**, in case a non-slip underlay should be used (e.g. AIREX® mat).

Appearance

Major emphasis is placed on functionality with ultrasoft products. A special manufacturing process is necessary to achieve softness and the wobble effect. Sometimes there are slight imperfections among these products, such as occasional surface bubbles, slight deviations in dimensional tolerances and slight differences in colour nuances. However, this affects in no way the function of these products.

Features

AIREX® closed-cell soft foams do not absorb any water and possess permanent surplus buoyancy in water. These extremely robust and wear-resistant materials lose pressure (as do balloons) when the cellular bodies are subjected to long periods of pressure. Pressure points generally slowly recover. AIREX® soft foams are treated with antimicrobial finishing for **permanent** and effective protection against bacteria

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and fungi.

Cleaning

Soiled pads are best treated with a soft brush and warm soapy water. Stains and other marks should not be treated with solvents, alcohol or stain remover. If disinfection is required, commercial disinfectants can be used, e.g. Merfen, Descogen or Chromium. AIREX® products should not be sterilized with hot steam (autoclaving) or hot water.

Storage

AIREX® Balance products are best stored **flat**. To avoid pressure, do not store any objects on top of them. Continuous pressure lasting for more than approximately 12 hours should be avoided. Storing them in bent fashion (e.g. hung over a washing line) will leave visible pressure points which will never completely recover. Constant storage in water or long periods of direct sunlight will shorten the life of the products, causing the surface to become brittle and stiff over time.

Hygienic

Simple to clean,
antimicrobial Sanitized® finishing.
(Biocidal substance: Folpet)

To lengthen product life, we recommend rinsing them after use with fresh water, shaking off excess water and allowing them to air dry (because of pressure points, never over a radiator, etc.). Damp products should

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never be stored in lockers or stacked in confined spaces.

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We hope you have a lot of fun and benefits with the products you have purchased. If you have any questions, please contact your dealer.

Vi håper at du vil ha god nytte av de produktene du har anskaffet og mye moro med dem. Hvis det er ting du fremdeles lurer på, ber vi deg henvende deg til din forhandler.

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Manufactured by: AIREX® AG, CH-5643 Sins/Switzerland

Distributed by: